

AireCore and Laminate Cabinets

Follow these guidelines to disinfect, clean, and care for Solaire Medical cabinets with laminated surfaces.

Cleaning Materials

- Use compounds that contain quaternary ammonium; they are the best choice for disinfecting and cleaning your cabinets.
- Use a 10%-50% isopropyl alcohol solution as a good option when quaternary ammonium compounds are not available.

Recommended disinfectant/cleaning products

- Virox® disinfectant cleaner
- Hepacide Quat® II virucidal disinfectant cleaners
- PDI® Sani-Cloth® cleaner
- CaviWipes™ disinfectant cleaner
- Dispatch® bleach germicidal cleaners